


SmartValve™

A Hysteroscope Accessory

Indications for Use and Sterilization Instructions

Indications for Use

SmartValve™, a hysteroscope accessory is a single-use per procedure disposable valve. The smart design of the valve prevents fluid splash-back through-out the procedure while allowing smooth insertion of devices. The valve is non-sterile and can be used with any 5-French or smaller device.

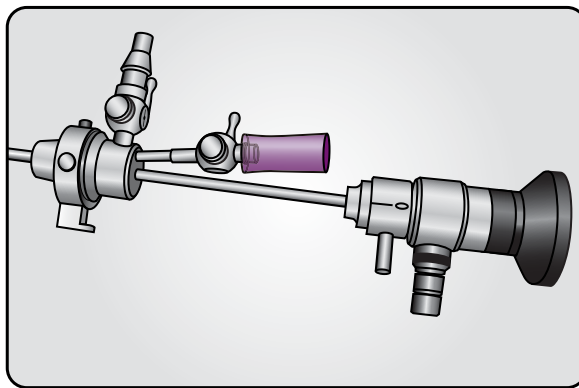

Sterilization and Disinfection Instructions

SmartValve is provided non-sterile. Please refer to the following sterilization and disinfection instructions.

- Sterilization: The valve can be sterilized using an autoclave at 121 degrees Celsius for 40 minutes.
- Disinfection: The valve can be disinfected using a 12-minute soak in Cidex® OPA followed by a water rinse, per the Cidex OPA IFU.

Note: It is normal for the color of the valve to get slightly darker after soaking in Cidex OPA. This does not affect the functionality of the valve.

Service and Support

Please contact your sales representative for additional information, or to place an order please call 1.800.442.9892 ext. 38540 and reference part # A10Valve.

Hologic, SmartValve A Hysteroscope Accessory and associated logos are trademarks and/or registered trademarks of Hologic, Inc. and/or its subsidiaries in the United States and/or other countries. All trademarks referenced herein are the property of their respective owners.

SmartValve A Hysteroscope Accessory is manufactured for and distributed by Hologic, Inc.

© 2010 Hologic, Inc. All rights reserved. Printed in USA.



MAN-01831-001 Rev. 001

